Side rail height extender

en	Bed accessory for Britt and Line Side rail Service Manual	1
da	Sengetilbehør til Britt og Line sengehest Servicemanual	2
de	Bettzubehör für Seitengitter Britt und Line Service Handbuch	3
es	Accesorio de cama para barandillas Britt y Line Manual de servicio.	4
fr	Accessoire de lit pour la barrière Britt et Line Manuel de maintenance	5
it	Accessorio per letto per sponda laterale Britt e Line Manuale per la manutenzione.	6
nl	Bedaccessoire voor het Britt- en Line-zijhekken Servicehandleiding	7
nn	Sengetilbehør for sengegrindene Britt og Line Servicehåndbok.	8
pt	Acessório de cama para a guarda lateral Britt e Line Manual de assistência.	9
sv	Sängtillbehör för Britt och Line sänggrind Servicemanual	11

I General

I.1 About this manual

This manual belongs to an Invacare accessory and it contains important information about handling and assembly. In order to ensure safety when using the product, read the user manual of the related product carefully and follow the safety instructions.

Symbols in this manual

In this user manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING
Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION
Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT
Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations
Gives useful tips, recommendations and information for efficient, trouble-free use.

2 Assembly

2.1 General

When this side rail height extender is used, the allowed mattress thickness can be increased by 15 cm. See user manual for your bed and side rail for correct mattress sizes.

- 1538345-0125 Side rail height extender for Line side rails, *Figure 1*.
- 1538420-0125 Side rail height extender for Britt V side rails, *Figure 2*.

WARNING!

Risk of falling out of bed if the side rail height extender loosens/falls off leading to body injuries or death.

- Make sure the correct side rail height extender is used (shape of attachments must match shape of side rail). See label; *Figure 1*, *Figure 2*, on your product to determine which side rail height extender to use for your specific side rail.
- Make sure that the side rail height extender is properly attached (finger screws tightened).




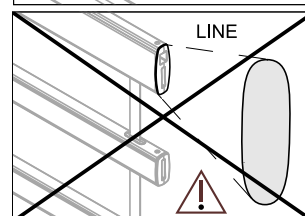
Figure 1

Figure 2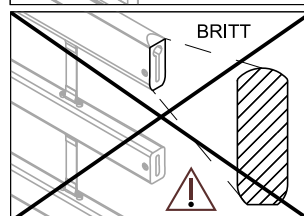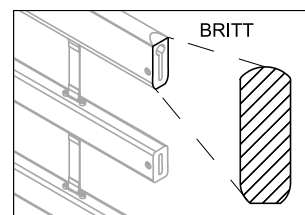

DEALER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.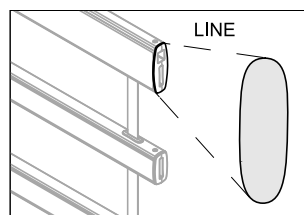

Yes, you can.®

2.2 Mounting of side rail height extender


WARNING!

Risk of squeezing or entrapment leading to body injuries or death.


- Make sure that the side rail height extender is placed centered on the side rail.
- Make sure that the distance between the bed ends and the outer edge of the side rail height extender is less than 6 cm.

1. Lock the side rail in its highest position.
- 2.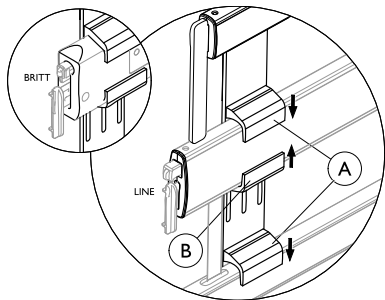


Hang the attachments **A** on the upper and middle bars of the side rail; attachments facing the inside of the bed.

3. Push the middle attachment **B** upwards onto the side rail bar, as close as possible.
- 4.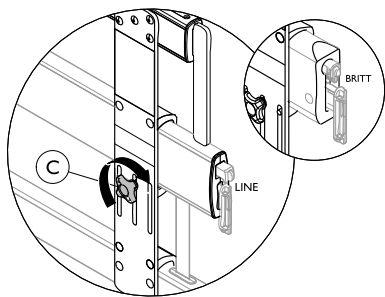

Tighten the finger screw **C** on the front.

5. Repeat step 3-4 in the other end of the side rail.
- 6.

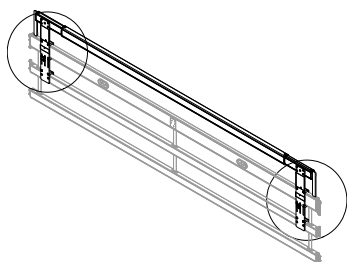Make sure the side rail height extender is tightly attached and does not feel loose.

3 Maintenance

3.1 Cleaning

For cleaning see user manual for your bed and side rail; section Cleaning and disinfection.

3.2 Waste disposal

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Waste disposal must comply with the laws and regulations pertaining waste handling in each country.

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Ziehen Sie die Flügelschraube © auf der Vorderseite fest.
Invacare AB
Invacare Rea AB