
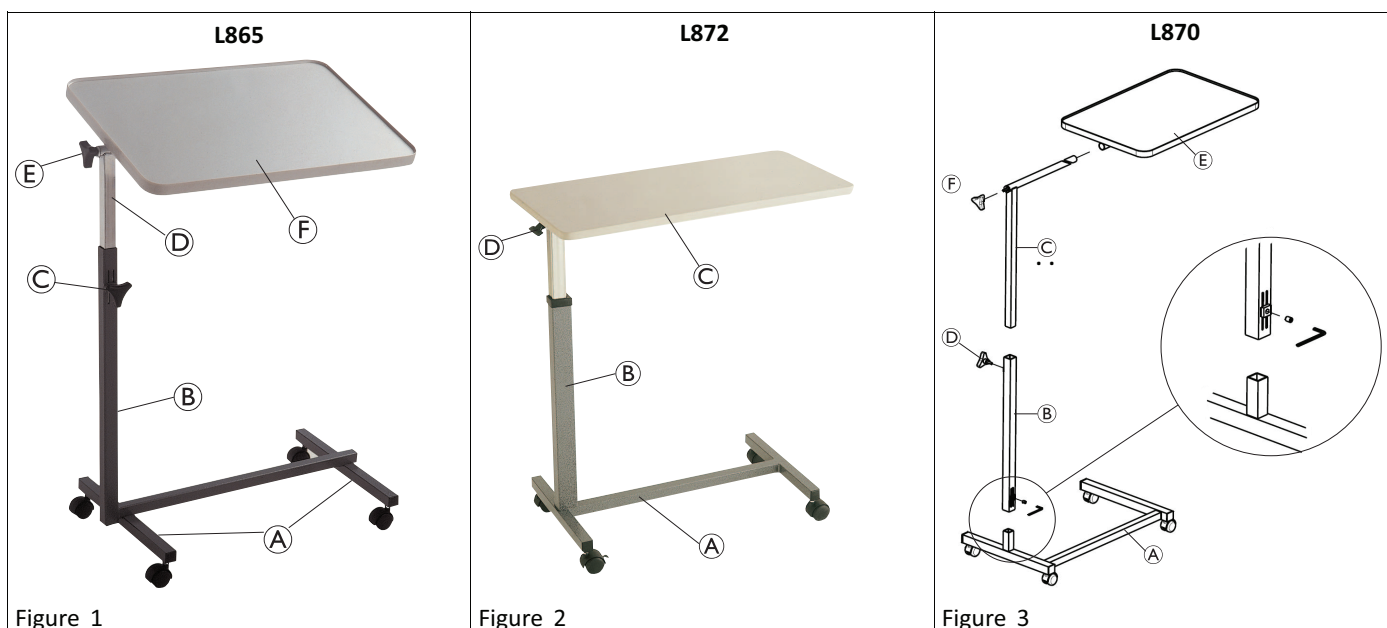
Invacare® Accessories

Invacare® Pausa (L865), Invacare® Kauma (L872), Invacare® Baya (L870)

en Bed side tables		it Vassoio letto a carrello	
User Manual	2	Manuale d'uso	7
da Sengeborde		lt Prie lovos statomi staliukai	
Brugsanvisning	2	Naudojimosi instrukcija	8
de Beistelltische		lv Galdiņi pie gultas	
Gebrauchsanweisung	3	Instrukciju rokasgrāmata	9
es Mesitas para cama		nl Bedtafels	
Manual del usuario	4	Gebruiksaanwijzing	10
fi Vuoteen sivupöydät		no Sidebord for senger	
Käyttöohje	5	Brugsanvisning	11
fr Tables de chevet		pt Mesas de cabeceira	
Manuel d'utilisation	6	Manual de utilização	12
hr Stolići uz krevet		sv Sängbord	
Upute za uporabu	7	Brugsanvisning	12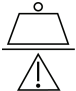

	L865	L872	L870
	10 kg	10 kg	10 kg

1 General

en

1.1 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

2 Safety

CAUTION! Risk of Injury

- Check that the product has been delivered in perfect condition before you use it.
- If you are unsure about usage, adjustment or maintenance of the product, contact your Invacare distributor.
- Do not exceed the maximum load, shown on the product label.
- Do not use or store the product outdoors.
- Periodically check stability and rigidity of the product and, where applicable, that the screws are tightened.

Cleaning and disinfection

- Remove any dirt or dust using a soft, damp cloth.
- Wipe surfaces thoroughly with a soft cloth slightly moistened with detergent solution until all visible soil is removed.
- Remove all detergent residue with a clean damp cloth.
- Dry with a soft cloth.
- For disinfection in hospital environments, follow the hygiene specialist's recommendations.

3 Assembly

3.1 Invacare®Pausa (L865)

See figure 1

1. Attach the two feet on castors (A) under the frame (B) using 2 screws.
2. Insert hand screw (C) into the upper section of the frame (B).
3. Slide the square tube of the tray support (D) into the vertical section of the frame (B), so that the tray support is located above the frame base.
4. Loosen hand screw (E) and slide the tray (F) on the tray support.
5. Adjust the tray to the desired inclination and tighten hand screw (E).
6. Adjust the height of the tray and tighten hand screw (C).

3.2 Invacare®Kauma (L872)

See figure 2

CAUTION!

The height adjustment system is controlled by a spring.

- Follow the instructions in this paragraph carefully and in the indicated order.

1. Install the rack (B) on the base (A):
 - a. Insert the two large screws in the two-hole plate, and insert the two small cylinders.
 - b. Place this assembly below the castor base and tighten the screws through the base into the rack. Make sure, the tray is located above the base.
2. Install the tray (C) by sliding it onto its support and fix it using two 8x 15 screws.
3. Install the adjustment system: Carefully unscrew the small black screw above the red label and replace it with the small grey screw supplied with the table. DO NOT remove the screw covered by the red label.

Adjusting the height

- To raise the tray, pull it upwards.
- To lower the tray, press the black lever.

3.3 Invacare®Baya (L870)

See figure 3

1. Install the square tube (B) on the base (A) using an Allen key.
2. Install the tray (E) on the tray holder tube and lock it with hand screw (F).
3. Slide the tray holder (C) into the square tube (B). Make sure, the tray is located above the base.
4. Adjust the height of the tray and tighten hand screw (D).

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Invacare AB