Invacare®

Styxo 2

en	Toilet frame	
	User Manual	3
cs	Toaletní konstrukce	
	Uživatelská příručka.....	6
de	Toilettengestell	
	Benutzerhandbuch.....	9
es	Silla de WC 3 en 1.....	12
fr	Cadre de toilettes	
	Manuel de l'utilisateur.....	15
it	Rialzo stabilizzante WC	
	Manuale utente.....	18
nl	Toilet frame	
	Gebruikershandleiding.....	21
no	Toalettramme	
	Brugerhåndbok.....	24
pt	Cadeira Sanitária	
	Manual do utilizador... ..	27
sv	Toalettstomme	
	Användarhandbok.....	30

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.

INVACARE  Hangzhou Dunli Medical Instruments Co.,Ltd No.1 Dunli Road,Renhe Street, Yuhang, Hangzhou, China, 311107

Styxo 2 Importer: Invacare GmbH, Am Achener Hof 8, 88316 Isny, Germany

MOD DL 3010064

UDI (01)06974396360021(10)YYMMDLXXXX

MD  Manufactured on: YYYY-MM-DD **159 kg**

CE **EC REP**

Sungo Europe B.V. Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Fig. 1

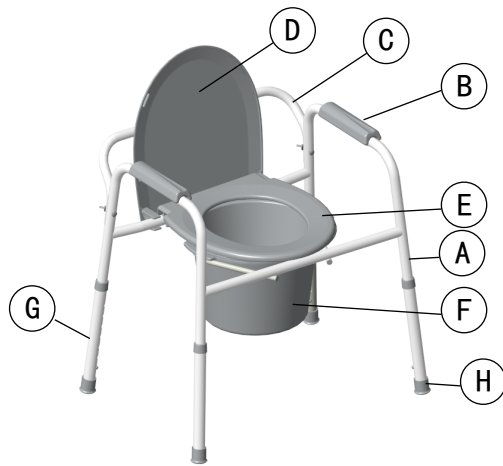
Fig. 2

Fig. 3

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific Standards

The product conforms to ISO 17966 (Assistive products for personal hygiene that supports users) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling!

- Do not use if defective.
- Do not make any unauthorised alterations or modifications to the product.
- Do not move or displace the product when under load or during use.
- All four legs must have full contact to the ground.
- Do not stand on this product.
- Use only indoors.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

The plastic parts of the Styxo 2 are not fire retardant. Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

6.2 Labels and Symbols on the Product (Fig. 1)

A	Manufacturer	F	Production date
B	European Conformity	G	Lot number
C	European Representative	H	Unique Device Identifier
		I	Medical Device
D	Read user manual	J	Model number
E	Maximum user weight	K	Product designation

The identification label is attached to the tubing on the left side.

7 Scope of Delivery (Fig. 2)

A	Frame, pre-assembled	E	Seat
B	Armrest pad, pre-assembled	F	Toilet pan with lid & Splash guard
C	Back tube	G	Leg
D	Lid	H	Foot

8 Intended Use

This product is intended to provide help for toilet tasks avoiding users to walk long distances to the toilet room. In its toilet frame configuration it helps the user to sit down on a standard toilet and getting up from it.

Users are any adults and adolescents with restricted or limited mobility (e.g.: legs and/or hips) or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

Indication

Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 159 kg.

9 Product Description

Invacare® Styxo 2 is a toilet frame acting like either a toilet chair or a toilet seat raiser. It is composed of four height adjustable feet, armrests, a toilet pan, a splash guard and a lid that can be removed.

10 Setup and Usage (Fig. 3)

Installing/Removing the Toilet Seat and Lid

- Before removing/installing seat and lid, allow seat/ring to reach room temperature.
 -  This will help prevent the seat clamps from breaking when being removed/installed onto commode back frame rail.
- Align the clamps of the toilet seat onto the back frame rail. (Each clamp should sit outside the pail holders.) Push down on seat until it clamps firmly into place.
- Repeat the same process for the lid.
- When removing the seat or lid, remove the lid first and then the seat. To remove, grasp the back of the lid and/or seat and lift up.

Height Adjustment

Commode's height is adjustable for use in an elevated position.

- Depress detent button on leg piece and move up or down to the desired position.
- Ensure that the detent button is fully engaged and leg does not move.

- Repeat steps 1 and 2 for the remaining three legs.

WARNING!

Risk of Falling

- All four legs must be adjusted to the same height.
- Ensure that the pins are correctly engaged after adjustment.

SITTING AND STANDING

- Backup to the commode until you feel the back of your knees touching it.
- Reach back for the armrests. Place both hands firmly on the armrests before sitting on the commode.
- Slowly lower yourself onto the commode by bending your elbows and, if possible, your hips and knees.
- Reverse steps 2 through 3 for getting up. Push on the armrests as much as comfortably possible. Make sure you have your balance before releasing your grasp from the commode armrests.

Over The Toilet Use

- Lift seat of commode and remove bucket.
- Replace with splash guard.
- Lift lid of toilet. Position commode over toilet with splash guard in bowl and lower commode seat.

11 Maintenance

11.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to. Your commode can be easily cleaned with regular household cleaning products. DO NOT use abrasive powders or scouring pads.

11.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

Cleaning Intervals

! IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Cleaning Instructions

! IMPORTANT!

- Clean by hand
- Maximum cleaning temperature 60 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

Lubrication

To make raising or lowering the seat/lid easier, you may lubricate the commode back frame rail with a very small amount of petroleum jelly or furniture polish.

12 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- **Inspection**
- Cleaning and disinfection

For detailed information, see *11 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

13 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

14 Technical Data

Dimensions	
Total height (closed lid)	58 – 73 cm (22.6"-28.6")
Total depth	48 cm (19")
Overall width	61 cm (24")

Seat height range	38-53 cm (15"-21")
Adjustment steps for seat height	2.54 cm (1")
Height, seat to armrest	20 cm (8")
Width between armrests	48 cm (19")
Seat depth	38 cm (15")
Width hygiene recess	20.5 cm (8.1")
Depth hygiene recess	24.5 cm (9.6")

Weights	
Weight	4.56 kg (10-lb.)
Maximum user weight	159 kg (350-lb.)

Materials	
Frame	Steel, powder coated
Seat / pan	PP
Armrest pad	PE-HD
Plug / Plastic rings on tubes	PP
Foot	TPU

NOTE: Feet on this product contain dry natural rubber.

NOTE: Device is powder coated to prevent corrosion.

15 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare AB
Hangzhou Dunli Medical instruments Co., Ltd